Bleichgel 6,1 %

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Version 05/2017

Bleaching 6,1%

Instructions for use for the dentist

Bleaching 6,1% is a bleaching gel for whitening discoloured teeth. It is suitable for minimally invasive treatment using a custom-made bleaching tray. Patients can apply the gel themselves after being instructed how to use it properly by the dentist. A syringe contains 6,1% hydrogen peroxide. One syringe contains 2 ml of material.

Applications

- Whitening abnormally discoloured teeth
- Whitening non-vital teeth
- Whitening teeth discoloured by medication

Contraindications

- Known allergy to or intolerance of any of the ingredients
- Pregnancy or breast-feeding
- Exposed cervical areas of the teeth, untreated caries
- Heavy smoking, unless the patient abstains from smoking during treatment

Preparation

- 1 Full case history
- 2 Examination of the teeth and soft tissue. Treat caries, leaking fillings, enamel cracks as well as exposed dentine provisionally beforehand to protect against pulp irritation.
- 3 Inform the patient of the options for whitening as well as the anticipated treatment period. The time required for treatment depends on the type of discolouration and tooth structure and varies from patient to patient. Generally two weeks are adequate, but the treatment period can extend to several weeks or months.
- 4 Professional scale and polish and removal of dental calculus
- 5 Take the existing tooth shade.
- 6 Take the impression, e.g. with alginate or elastomer impression material.

Note: Separate treatment of each jaw (e.g. first the upper and then the lower jaw) gives patients a greater sense of achievement, as they can see the changes immediately. This is, however, very time-consuming.

Fabricating the vacuum-formed tray

An optimally fitting vacuum-formed tray is crucial for successful whitening and reduces the risk of the patient suffering gingival irritation.

- 1 Fabricate the stone model and trim. Remove plaster bubbles.

Version 05/2017

- 2 After the model is completely dry, apply block-out resin. This is applied to the labial surfaces of the teeth to be whitened and is used later as a reservoir for the gel. Usually this involves teeth 15 – 25 and 35 – 45. It is of course possible to prepare a smaller or larger number of teeth or single teeth. Important: Apply the block-out resin in layers approx. 0.5 mm thick and maintain a gap of approx. 0.5 – 1 mm to the gingival margin. Do not cover incisal edges and occlusal surfaces with gel!
- 3 Polymerise the block-out resin. A standard halogen polymerisation lamp (approx. 20 sec. per tooth) or a light-curing unit (approx. 2 – 3 min., depending on the manufacturer) can be used for polymerising.
- 4 After curing, completely remove the separating agent on the surface using a cloth moistened with alcohol. Do not use acetone or similar cleaning agents.
- 5 Vacuum form over the prepared stone model in the vacuum-forming unit using the foil supplied.
- 6 Remove the vacuum formed foil from the model after cooling and cut away the excess.
- 7 Then cut accurately with fine scissors ensuring the bleaching tray finishes approx. 0.5 mm above the gingival margin. Relieve the interdental papillae to avoid subsequent irritation.
- 8 Replace the bleaching tray on the model to check that it fits correctly. Smooth any sharp edges. Heat any protruding sections briefly (e.g. with a burner) and correct the fit by pressing down with a moistened finger.
- 9 Clean the bleaching tray thoroughly.

Instructing the patient

- 1 Check the retention and fit of the bleaching tray in the patient's mouth.
- 2 Demonstrate how to fill the tray with Bleach 'n Smile Home using a syringe. Do not overfill with material. One syringe is adequate to fill the upper and lower bleaching trays at least once.
- 3 Insert the bleaching tray and remove excess material using a toothbrush or finger. Demonstrate this procedure as well.
- 4 The treatment period should last 2 – 3 hours per day and the bleaching tray can also be left in the mouth overnight.
- 5 The patient should not eat, drink or smoke during treatment.
- 6 Remove the bleaching tray and clean thoroughly using warm water and a toothbrush. Dry well and store in the storage box. Rinse the teeth.
- 7 Ensure that the patient is aware that consuming anything strongly acidic (Cola, citrus drinks and fruit) or strongly coloured (coffee, tea, red wine) during this period may impair the success of the treatment.
- 8 Give the patient the syringes with bleaching gel, bleaching trays, storage box and the instructions for use together with the case.
- 9 Arrange a recall appointment.

Version 05/2017

Recall

- 1 Check after 2 to 3 days whether the patient has carried out the treatment according to the instructions. Check the condition of the gingiva and bleaching tray.
- 2 Give the patient a refill set.
- 3 Check again after approx. 2 weeks and compare the shade with the initial shade. Continue treatment if necessary.
- 4 Fluoridation should be carried out on completion of treatment.
- 5 Instruct the patient to keep the bleaching trays for bleaching the teeth again at a later date, if required.

Information

Filling materials as well as crowns, bridges and veneers made from porcelain, composite or metal are not bleached. They should be replaced if necessary and matched to the new, whiter tooth shade. There should be a period of 2 weeks between bleaching and renewing any restorations.

In certain cases there can be hypersensitivity of the teeth and mucosal irritation during treatment; these symptoms disappear if the amount of bleaching gel is reduced or treatment is discontinued.

White spots may appear on the teeth during treatment. These are caused by transient demineralisation and usually disappear again.

Caution

Bleach Home contains carbamide peroxide, glycerine, carboxymethyl-cellulose and flavouring. Avoid contact with the eyes, skin and mucosal tissue. On contact, rinse with water and, on contact with the eyes, also consult a doctor.

If an excessive amount of gel is swallowed, it can cause heartburn or a feeling of nausea in sensitive patients. Keep out of reach of children.

Storage

Store at 3-9°C in a cool place (refrigerator), but do not freeze. Protect from heat and direct sunlight. Do not use after the expiry date.

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