C+B Opaquer

CE 0297

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Instructions for use

CE 0297

The framework is prepared with the metal-resin bonding system Metal Bonder.

C+B Opaquer is applied with a stiff, short-haired brush. Incomplete opaque coverage negatively affects the shade. At least two coats are required to ensure complete coverage. The flow behaviour of the opaque is improved by mixing it on a mixing pad. The paste should be applied especially thinly to retention areas. Each layer is polymerized separately.

| | |
|------------------------------|--------|
| Spektra LED Incisal /Dentin: | 30 sec |
| Cervical: | 1 min |
| Spektra fast / Spektra 2000: | 3 min |

If the opaquer layers are applied too thickly, it prevents optimum polymerization and weakens the bond. Extending the polymerization time does not increase the depth of polymerization.

Useful tips

The surfaces of the framework should be cleaned of any grease and polish residue. The opaquer should only be applied in very thin layers.

Working time for the materials

Depending on the lighting conditions 1- 3 min.

Storage

The material should be stored at 10- 25° C. Close the syringe properly immediately after use and protect against the effects of direct light.

Shelf life

The maximum shelf life is printed on the label of each syringe. Do not use after the expiry date.

Each syringe contains: 3 g

Guarantee

Our technical instructions, regardless of whether they are provided verbally, in writing or during practical demonstrations, are based on our own experience and should only be considered guidelines. As our products are subject to continued development, we reserve the right to modify them.

Note

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (e. g. allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

Contra-indications / interactions

If a patient has known hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient’s mouth before using the product.

| Polymerization times | |
|--------------------------|--------------|
| Light-curing unit | Paste opaque |
| Spektra LED | 1 min. |
| Spektra 2000 fast | 3 min. |
| Spektra 2000 | 3 min. |
| DC-XS | 90 sec. |
| Uni-XS / HiLite | 90 sec. |
| Spektramat | 2 min. |
| GC Labolight LV-II / III | 1 min. |
| Solidilite EX / V | 1 min. |

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GDF Gesellschaft für dentale Forschung und Innovationen GmbH