# Pattern Acryl

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## Instructions

### Indication

Building up patterns of outer copings with a brush

### Contraindication:

If the user is known to be allergic to one of the constituents of this material, it must not be used.

### Side-effects:

None known to date.

### Properties:

- Burns out completely
- Minimal shrinkage

### Processing:

- Dispense the required amounts of liquid and powder into separate mixing vessels.
- Dip the tip of the brush into the liquid and then pick up a small amount of powder.
- The resin swells on the brush tip and can be placed and shaped exactly on the

coping.

- Repeat this procedure several times until the outer coping has been built up.
- Carefully raise the pattern from the inner coping and check the marginal fit as well as the inside of the pattern.
- Replace the pattern on the inner coping and use a cutter to trim it to uniform thickness.

### Please note:

- If this material is being used on a plaster model, FuturaSep Plus separating agent should be applied.
- Clean the brush with Pattern Acryl liquid after use or, if necessary, during use.
- Do not pour any leftover liquid or powder back into the bottle.

### Colour:

red

**Storage:**

Store at room temperature (21°C) and do not expose to direct sunlight.

**Presentation:**

100 g powder  
100 ml liquid  
1 Set: 100 g powder and 100 ml liquid

**Contains methacrylates**

May cause an allergic skin reaction.

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