

C&B Composite PASTE & FLOW

Light-curing crown and bridge lab composite for aesthetic veneering. The material comes in two different consistencies and leaves total freedom of choice between liquid and paste. The system combines strength, aesthetics and versatility.

Features	Benefits
All shades in paste and flow	Almost unlimited combinations and easy handling, finishing, polishing
8 Master-Denties, 4 Incisals	Simple system for the daily work in the dental lab
Ceramic-like properties	Very convenient to process
Extremely abrasion-resistant	Long-lasting restorations
Excellent polishability	Lowplaque affinity
Reduced transparency	Very good opaque colors



Shades: See color scheme below
 Packaging size: 5 g syringe paste, 3 g syringe flow
 (10 application tips recommended)

Technical data:

	Paste	Flow
Hardness	400 MPa	370 MPa
Flexural strength	95 MPa	140 MPa
E-Module	6900 MPa	6500 MPa
Compressive strength	360 MPa	360 MPa

Version 05/2016

C&B Composite Paste & Flow color scheme (Paste + Flow)			
Color according to Vita®*	paste opopaque	dentine	incisal
A1	light	A1 / B1	S58
A2	light	A2 / B2	S58
A3	light	A3	S59
A3,5	medium	A3,5	S59
A4	medium	A4	S60
B1	light	A1 / B1	S57
B2**	medium	A2 / B2	S58
B3	light	B3 / B4	S59
B4	medium	B3 / B4	S59
C1**	medium	C1 / D2 / D3 / D4	S59
C2	medium	C2 / C3 / C4	S59
C3	dark	C2 / C3 / C4	S59
D2	light	C1 / D2 / D3 / D4	S60
D3	dark	C1 / D2 / D3 / D4	S59
Effect			blue
Effect			red
Effect			clear
Effect			light

** Vita is a registered trade mark of Vita Zahnfabrik H. Rauter GmbH & Co. KG, Bad Säckingen.*

** Remarks	
B2	Take incisal material along the edges all the way down to the shoulder. Apply incisal only relatively thin onto the shoulder area of the restoration
C1	Completely coat the body of the restoration with incisal material

Version 05/2016